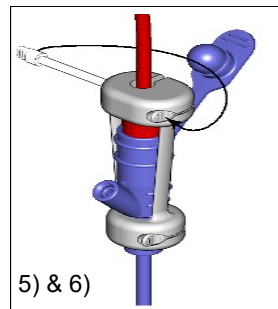
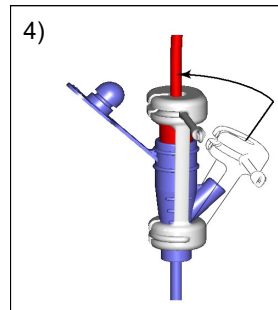
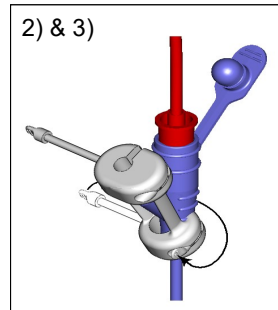
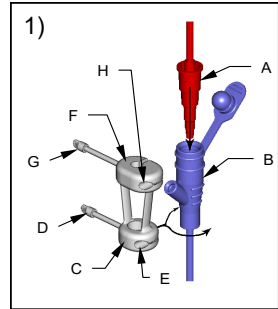
AMT Clamp™

Feeding Tube Connector

CE

- ES** AMT Clamp™ Conector del tubo de alimentación.
- DE** AMT Clamp™ Ernährungs sondenadapter.
- IT** AMT Clamp™ Connettore per tubo di alimentazione.
- FR** AMT Clamp™ Connecteur de tube d'alimentation.
- NO** AMT Clamp™ Enterale ernærings slanger kontakt.
- SV** AMT Clamp™ Matningsrörskontakt.
- PT-BR** AMT Clamp™ Conector do tubo de alimentação.
- NL** AMT Clamp™ Connector voor voedings slang.
- DA** AMT Clamp™ Sammenkobling til ernærings sonde.
- FI** AMT Clamp™ Ruokintaletkun liitin.
- AR** AMT Clamp™ وصيلة أنبوب التغذية.
- BG** AMT Clamp™ Конектор на тръба за хранене.
- CS** AMT Clamp™ Konektor vyživovací sondy.
- ET** AMT Clamp™ Toitmissondi konnektor.
- EL** AMT Clamp™ Σύνδεσμος σωλήνα σίτισης.
- HU** AMT Clamp™ Tápláló cső csatlakozója.
- JA** AMT Clamp™ 栄養チューブコネクタ。
- LV** AMT Clamp™ Barošanas caurules savienotājs.
- LT** AMT Clamp™ Maitinimo vamzdelio jungčiai.
- PT** AMT Clamp™ Conector de sonda de alimentação.
- PL** AMT Clamp™ Złącze zgłębnika.
- RO** AMT Clamp™ Tub de hrănire connector.
- SK** AMT Clamp™ Konektor hadičky na podávanie výživy.
- SL** AMT Clamp™ Priključek za cevko za hranjenje.
- TR** AMT Clamp™ Beslenme tüpü bağlantı parçası.

Made in USA

C3646-G 4/2020

MD Rx ONLY

2

~~LATEX~~

~~PHI~~
DEHP

NON STERILE

MR

AMT Clamp™ Feeding Tube Connector

INSTRUCTIONS FOR USE

INDICATIONS, CONTRAINDICATIONS, DEVICE LONGEVITY

Intended Use

The AMT Clamp™ is intended to maintain the connection between feeding set insertion points.

Indications for Use

The AMT Clamp™ is indicated for use in patients of all ages with a history of their feeding set becoming disconnected from their feeding device.

Contraindications for Use

The AMT Clamp™ is contraindicated for use in patients with a history of their feeding tubes being pulled out.

Device Longevity

The AMT Clamp™ is meant to be periodically replaced for optimal performance, functionality, and cleanliness. AMT recommends that the clamp be changed at least every 4 weeks or as often as indicated by your healthcare professional. Device performance and functionality can degrade over time depending on usage and environmental conditions. Some factors that can lead to reduced longevity include: frequency of use, trauma to the device, and overall care of the AMT Clamp™.

The AMT Clamp™ should be replaced if tears or other signs of failure are noted. The device can be placed in conventional waste for disposal.

Note: Please contact AMT, our Authorized Representative (EC Rep), and/or the competent authority of the member state in which you are established if a serious incident has occurred in relation to the device.

CLINICAL BENEFITS, PERFORMANCE CHARACTERISTICS

Clinical Benefits

Clinical benefits to be expected when using the AMT Clamp™ include but are not limited to:

- Prevents feeding set from becoming disconnected from feeding device
- Ensures the patient receives the essential nutrients while avoiding the mess of lost formula.

Performance Characteristics

Performance characteristics of the AMT Clamp™ include but are not limited to:

- Soft, comfortable, no slip material
- Allows multiple removal and placement cycles
- Compatible with various manufacturers' enteral feeding tubes, christmas tree adapters, y-ports and g-tubes.

PLACING THE AMT CLAMP™

Note: This device is intended to be used by qualified professionals and lay users.

1. Make sure the christmas tree adapter (A) and the feeding set/G-tube (B) are connected securely per original manufacturer's directions.
2. Place the small bottom ring of the AMT Clamp™ (C) around the base of the feeding set/G-tube (B) just *below* the side port.
3. Secure the wrap-strap (D) around the bottom ring of the AMT Clamp™ by tucking the strap into the slot and placing the tab into pocket (E).
4. Stretch the main straps so the large top ring (F) is above the christmas tree adapter (A) and snap the top ring around the tubing.
5. Secure the wrap-strap (G) around the top ring of the AMT Clamp™ by tucking the strap into the slot and placing the tab into pocket (H).
6. Ensure your feed set connection and AMT Clamp™ is not bent and all parts appear to be in their proper places before you begin enteral feeding.

WARNING: Excessive tension may inadvertently remove the feeding device from the patient thereby disrupting the gastrostomy site, which could result in tissue necrosis, infection, peritonitis, sepsis, and associated sequelae.

WARNING: Improper placement of the AMT Clamp™ may result in occlusion of the feeding device.

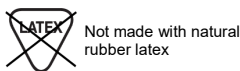
WARNING: If unable to replace feeding device easily, contact your physician immediately.

Note: Feeding system should be checked every two hours (or as needed) to ensure safe delivery of nutritional formula.

Not made with DEHP (Di (2-ethylhexyl)Phthalate)

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Applied Medical Technology, Inc.