
AMT Balloon Stoma Measuring Device

CE
2862

- ES** Dispositivo de Globo de Medición de Estoma AMT.
- DE** Ballonspitze der AMT Ballon-Stoma-Messvorrichtung.
- IT** Dispositivo di misurazione foro AMT con palloncino.
- FR** L'appareil de mesure de stomie à ballon AMT.
- NO** AMT-ballong stoma målingsutstyr.
- SV** AMT:s stomamätare med ballong.
- PT-BR** Dispositivos de medição de estômago de balão AMT.
- NL** AMT ballonmeter voor stoma's.
- DA** Ballonspidsen på AMT ballon stoma måleinstrumentet.
- FI** AMT Balloon Stoma -mittauslaitteiden .
- AR** جهاز قياس الفقرة الباليونية AMT
- BG** Балонно устройство за измерване на стома на AMT.
- CS** Balónkové zařízení k měření stomie AMT.
- ET** AMT balloonia stoomimõõtmisseade.
- EL** AMT Συσκευή Μέτρησης Στομίας Μπαλονιού.
- HU** AMT ballon sztómamérő készülék.
- JA** AMTバルーン・ストーマ測定デバイス.
- LV** AMT Balona stomas mērīšanas ierīce.
- LT** AMT balioninis stomos matavimo įtaisas.
- PL** Urządzenie pomiarowe AMT Balloon Stoma Measuring Device.
- PT** Aparelho de Medição de Estoma com Balão da AMT.
- RO** Dispozitiv de măsurare stomă cu balon AMT.
- SK** Zariadenie na meranie v mieste stómie s balónikom AMT.
- SL** AMT za merjenje stom z balončkom.
- TR** AMT Balonlu Stoma Ölçüm Cihazı.
- MS** Peranti Pengukuran Stoma Belon AMT.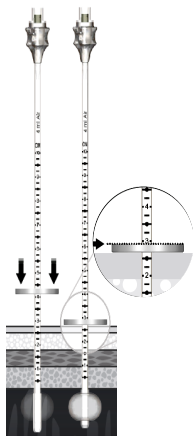

DEHP

R_x ONLY
STERILE EO

Made in the U.S.A.
C3702-H 04/2024

CAUTION: Federal (USA) law restricts this device to sale, distribution, and use by or on the order of a physician.

INTENDED USE

The AMT Balloon Stoma Measuring Device is intended to be used as an effective tool to measure the length of a gastrointestinal stoma. It is intended to be used for patients of all ages (infant to elderly).

INDICATIONS FOR USE

The AMT Balloon Stoma Measuring Devices are to be used by medical professionals and home health givers in association with replacement gastrostomy feeding devices. The purpose of the AMT Balloon Stoma Measuring Devices is to aid in determining the depth (length) of an established stoma tract in order to assist in determining the proper length replacement gastrostomy feeding device to be used. Minimum French Size is 10FR.

CONTRAINDICATIONS FOR USE

At this time there are no contraindications associated with the use of the AMT Balloon Stoma Measuring Device.

COMPLICATIONS

At this time there are no complications associated with the use of the AMT Balloon Stoma Measuring Device.

NOTE: Please contact AMT, our Authorized Representative (EC Rep), and/or the competent authority of the member state in which you are established if a serious incident has occurred in relation to the device.

CLINICAL BENEFITS

Clinical Benefits to be expected when using the AMT Balloon Stoma Measuring Device include but are not limited to:

- Increases the probability that a patient will receive a properly sized low profile gastrostomy tube allowing for optimal channeling of nutrition and medication
- Reduces potential formation of granulation tissue and other skin issues resulting from a feeding tube which is too tight or too loose/ leaking
- Reduces potential of inadvertent tube dislodgement which would necessitate replacement

PERFORMANCE CHARACTERISTICS

Performance Characteristics of the AMT Balloon Stoma Measuring Device include but are not limited to:

- Provides a means of accurately measuring the stoma tract length
- Helps to ensure proper fit, comfort, and safety of replacement gastrostomy feeding device
- Compatible with stoma tracts of varying diameter and length corresponding to common sizes of gastro-intestinally placed feeding tube devices

WARNING: THIS DEVICE IS INTENDED FOR SINGLE USE. DO NOT REUSE, RE-STERILIZE OR REPROCESS THIS MEDICAL DEVICE. DOING SO MAY COMPROMISE BIOCOMPATIBILITY CHARACTERISTICS, DEVICE PERFORMANCE AND/OR MATERIAL INTEGRITY; ANY OF WHICH MAY RESULT IN POTENTIAL PATIENT INJURY, ILLNESS AND/OR DEATH.

CAUTION: Please inspect all contents of the kit for damage. If the package is damaged or sterile barrier is breached, do not use the product. If any kit component is missing or damaged, discard and obtain another device.

INSTRUCTIONS

1. Open the AMT Balloon Stoma Measuring Device package and inspect the parts. Verify balloon inflates prior to using. If any damage is noticed, discard and obtain another device for use.
2. Slide the measuring disc up the tube - away from the balloon. If the disc does not slide on the tube, discard and obtain another device for use.
3. Remove the existing feeding tube from patient according to the original device's instructions for use. Work as quickly as possible to avoid spontaneous closure of the stoma.
4. Lubricate the balloon tip of the AMT Balloon Stoma Measuring Device using a water soluble lubricant.
5. Gently slide the tube portion of the AMT Balloon Stoma Measuring Device through the stoma site and into the stomach
6. Insert a luer-slip syringe containing 4 ml air into the fill valve, inflate the balloon, then remove the syringe.
7. Pull gently on the AMT Balloon Stoma Measuring Device until you can feel the balloon against the inside stomach wall.
8. Slide the measuring disk until it rests on the outside of the abdomen.
9. Read the measurement value on the tube above the disc (side farther from balloon) to determine the stoma tract length. The measuring accuracy of the device is +/- 0.2cm.

CAUTION: An accurate measurement of the stoma length is critical for safety and comfort of the patient. An inappropriately sized device can cause necrosis, buried bumper syndrome, and/or hypergranulation tissue.

10. Reinsert the syringe into the fill valve to deflate the balloon and remove the device.

NOTE: The device can be disposed of by following local disposal guidelines, facility protocol or through conventional waste.

11. Refer to the manufacturer's Directions For Use for the selection of the replacement feeding tube and instructions for placement.

THANK YOU! Thank you for choosing AMT. For additional help and information regarding the use of our device, feel free to contact AMT with the contact information on the back of the instructions for use. We are happy to hear your thoughts and help with your concerns and questions.

 Rx ONLY Prescription Only	 STERILE EO Sterilized Using Ethylene Oxide
 LATEX Not made with natural rubber latex	 DEHP Not made with DEHP (Di(2-ethylhexyl) Phthalate)
 2 Do not resterilize	 Do not use if package is damaged and consult instructions for use
 2 Single Use Only	 MD Medical Device

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© 2024 Applied Medical Technology, Inc.
Applied Medical Technology, Inc.