Patient Information Leaflet Capsule Monarch® and Capsule Dome Pre-loaded Replacement Gastrostomy Tube

ENGLISH

What is in this leaflet?

This leaflet will answer some questions about the Capsule Monarch® and Capsule Dome G-Tube. AMT has provided this information as an educational resource tool. This is not intended as a substitute for professional medical care. Your FIRST source of information should be your healthcare provider.

What is the Capsule Monarch® or Capsule Dome G-Tube?

The Capsule Monarch® and Capsule Dome G-Tube are gastrostomy feeding tubes that are inserted directly into the stomach through an opening known as a stoma. The devices include an encapsulated bolster with a self-lubricating capsule for a smooth insertion. An easy-to-use pull-away tether allows the capsule to split open and deploy the bolster internally. The capsule material falls away, dissolves rapidly, and passes. The internal silicone retention bolster helps secure the device inside the stomach, while a sliding external bolster helps maintain tube position. The sliding external bolster is adjustable to accommodate various stoma tract lengths. The Capsule Monarch® and Capsule Dome are available in various French sizes, with either a Legacy or ENFit® connector.

Figure 3: Capsule Technology

The Capsule Monarch® and Capsule Dome are made from medical-grade silicone (65%), medical-grade thermoplastic (5%), medical-grade silicone with a medical-grade thermoplastic barb (29%), and medical-grade silicone pad print ink (1%). The devices also contain a medical grade cellulose capsule and polyethylene suture that are no longer present after the device is placed. There are no manufacturing residuals that could pose a risk to the patient.

What is the Capsule Monarch® or Capsule Dome G-Tube Used for?

The Capsule Monarch® or Capsule Dome G-Tube is to be used as a percutaneous replacement gastrostomy tube for a patient with a well established gastrostomy tract. This device will assist in providing nutrition directly into the stomach through an established stoma in a patient who is unable to consume nutrition by conventional means. The Capsule Monarch® or Capsule Dome G-Tube can also deliver medication and allow for decompression of the stomach.

How do you use the Capsule Monarch® or Capsule Dome G-Tube after it has been placed?

Feeding: Open the cap to the port and connect the feed set to the tube. Deliver nutrition according to your healthcare professional's instructions. Make sure not to over-tighten the connection to the tube and ensure any clamps on the feed set are opened before starting to feed.

Flushing: Use room temperature water for tube flushing. The amount of water will depend on the patient's needs, clinical condition, and type of tube, but the average volume ranges from 10 to 50 ml for adults, and 3 to 10 ml for infants. Flush the feeding tube with water every 4-6 hours during continuous feeding, anytime the feeding is interrupted, before and after every intermittent feeding, or at least every 8 hours if the tube is not being used. Flush the feeding tube before and after medication channeling and between medications. Use a 30 to 60 ml syringe. Do not use smaller size syringes as this can increase pressure on the tube and potentially rupture smaller tubes. Do not use excessive force to flush the tube. Excessive force can perforate the tube and can cause injury to the gastrointestinal tract.

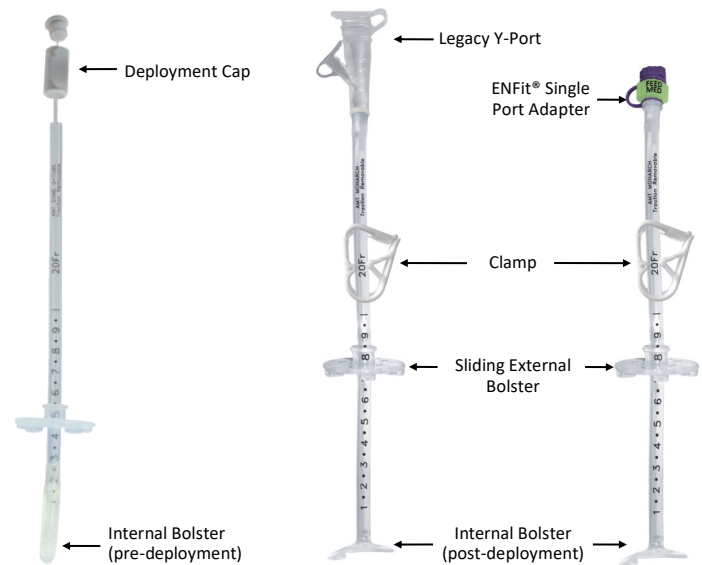
Figure 1: Capsule Monarch® G-Tube

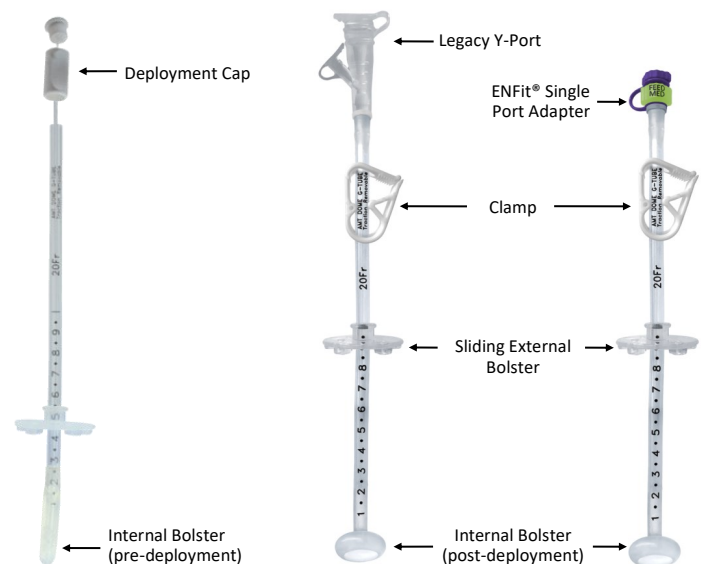
Figure 2: Capsule Dome G-Tube

Medication Channeling: Use liquid medication when possible and consult the pharmacist to determine if it is safe to crush solid medication and mix with water. If safe, pulverize the solid medication into a fine powder form and dissolve the powder in water before channeling through the feeding tube. Never crush enteric coated medication or mix medication with formula. Using a syringe, flush the tube with the prescribed amount of water.

Decompression: Decompression can be performed if there is a need to check for residuals, or if the patient tends to retain air during feeding causing bloating and discomfort. To decompress, open the port and drain stomach contents into a container. After decompression is complete, flush with water.

How do you take care of the Capsule Monarch® or Capsule Dome G-Tube after it has been placed?

Device Care: Check the device daily for any signs of damage or clogging. Clogging and/or reduced flow are indications of diminished performance. The stoma area should be kept clean and dry at all times. It is important to clean the stoma site every day. A cotton swab or terry cloth can be used to clean the skin around the device with mild soap and water. Clean the stoma as directed by your healthcare professional. The device should be rotated daily for site hygiene. Always check the stoma site for redness, pain/soreness, swelling, or any drainage. If any of these symptoms are observed, contact your healthcare professional for advice.

Clogging: First check to make sure that the tube is not kinked or clamped anywhere. If there is a visible clog in the tubing, attempt to massage the device to break up the clog. Connect a 30 to 60 ml syringe filled with warm water into the appropriate adapter or lumen of the tube and gently push and pull the syringe plunger to free the clog. It may take several cycles of pushing / pulling the plunger to clear the clog. If clog cannot be removed, contact your healthcare professional, as the tube may need to be replaced. Do not use excessive force or pressure to attempt to clear the clog. This can cause the tubing to rupture.

How long does the Capsule Monarch® or Capsule Dome G-Tube last?

Feeding devices are meant to be periodically replaced for optimal performance, functionality, and cleanliness. Device performance and functionality can degrade over time depending on usage and environmental conditions. Typical device longevity will vary for each patient depending on a number of factors, with typical device longevity ranging from 3-9 months. Some factors that can lead to reduced longevity include: gastric pH, diet of the patient, medications, trauma to the device, contact with sharp or abrasive objects, and overall tube care.

For optimal performance, it is recommended that the Capsule Monarch® or Capsule Dome G-Tube device be changed at least every 6 months or as often as indicated by your healthcare professional. Proactive replacement of the device will help ensure optimal functionality and will help prevent unexpected failure.

What are the possible side effects of having the Capsule Monarch® or Capsule Dome?

Potential complications when using the Capsule Monarch® or Capsule Dome include but are not limited to: Aspiration • Abscess, wound infection and skin breakdown • Hypergranulation tissue • Buried bumper syndrome • Pressure necrosis • Gastrointestinal bleeding and/or ulcerations • Ileus or gastroparesis • Intraperitoneal leakage • Bowel and gastric volvulus • Peritonitis • Gastrocolic fistula • Sepsis • Obstruction

Please consult your healthcare professional if you experience any of the following: Nausea, vomiting, abdominal bloating, or diarrhea • Pain, bleeding, and/or inflammation at the G-tube site • Crusting at the stoma site • Skin around the stoma site that is red, discolored, or raw • Stoma site drainage and/or puss that is white, yellow, or green and may smell bad • Repetitive leakage of food or stomach contents • Distended stomach • Fever • Tube clog • Migration of the tubing resulting in the device extending longer or shorter from the stoma • Extubation resulting in the device becoming removed from the stoma and unable to be replaced easily • Distinct indentation at the G-tube site or a distinct gap between the device and skin

There is a risk that the device may be pulled from the stoma if excessive force was applied to device during use. There could be leakage of gastric content around the tube or leakage from the device. The tube could become clogged or have reduced flow. The device may become discolored over days to months of use.

Is the Capsule Monarch® and Capsule Dome MRI Compatible?

The Capsule Monarch® and Capsule Dome are considered MR Safe once placed.

How do you report adverse events?

- For Australian Users: Notice that any serious incident that occurs in relation to the device should be reported to Applied Medical Technology, Inc. and The Therapeutic Goods Administration (TGA) at <http://www.tga.gov.au>.
- Please contact AMT, our Authorized Representative (EC Rep), and/or the competent authority of the member state in which you are established if a serious incident has occurred in relation to the device.

Product Codes:

Capsule Monarch® (Legacy)	Capsule Monarch® (ENFit®)	Capsule Monarch® with Guidewire (Legacy)	Capsule Monarch® with Guidewire (ENFit®)
9-C1210	9-C1210-I	9-CG1210	9-CG1210-I
9-C1410	9-C1410-I	9-CG1410	9-CG1410-I
9-C1810	9-C1810-I	9-CG1810	9-CG1810-I
9-C2010	9-C2010-I	9-CG2010	9-CG2010-I
Capsule Dome (Legacy)	Capsule Dome (ENFit®)	Capsule Dome with Guidewire (Legacy)	Capsule Dome with Guidewire (ENFit®)
9-C2020	9-C2020-I	9-CG2020	9-CG2020-I
9-C2420	9-C2420-I	9-CG2420	9-CG2420-I

Applied Medical Technology, Inc.

8006 Katherine Boulevard

Brecksville, OH 44141 USA

Toll Free: +1-800-869-7382 | Telephone: +1-440-717-4000

Website: www.appliedmedical.net | E-mail: cs@appliedmedical.net

© 2025 Applied Medical Technology, Inc. Rev C4751-C 07/2025

Patents: <http://www.AppliedMedical.net/Patents/> | Trademarks: <http://www.AppliedMedical.net/Trademarks/>

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