What is in this leaflet?

This leaflet will answer some questions about the NutraGlide® Nasal Feeding Tube. AMT has provided this information as an educational resource tool. This is not intended as a substitute for professional medical care. Your FIRST source of information should be your healthcare provider.

What is the NutraGlide®?

The NutraGlide® is a nasal feeding tube that is inserted into a patient's naris to provide feeding access to the stomach or small intestine. The NutraGlide® features a unique increasing internal diameter and open distal tip, which is designed to reduce clogging. The reverse taper tube design also makes it easier to remove a clog, if one should form. The device is available in various French sizes and lengths to accommodate the needs of patients of all age groups. All sizes feature AMT's proprietary Ready Glide™ coating, which eliminates the need for water activation, and is intended to provide an easier placement and removal of the stylet.

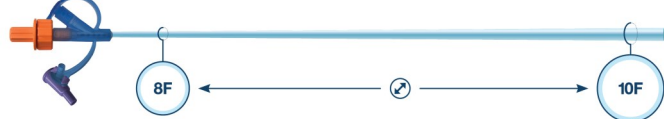
Figure 2: Reverse Taper Design

The NutraGlide® is made from medical-grade elastomeric thermoplastic, non-PVC (78% - 82%), medical-grade thermoplastic (16% - 20%), medical-grade adhesive (1%), medical-grade surface treatment (1%), and laser-engraved onto the medical-grade elastomeric thermoplastic tubing (<1%). There are no manufacturing residuals that could pose a risk to the patient.

What is the NutraGlide® Used for?

The NutraGlide® allows patients who cannot eat to receive nutrition. The device is intended to deliver enteral nutrition, medication, and fluid to neonatal, pediatric and adult patients via the naso-gastric or naso-intestinal route.

How do you use the NutraGlide® after it has been placed?

Feeding: Open the cap to the port and connect the feed set to the tube. Deliver nutrition according to your healthcare professional's instructions. Make sure not to over-tighten the connection to the tube and ensure any clamps on the feed set are opened before starting to feed.

Medication Channeling: Liquid medication is preferred when available. If solid medication is required, consult with your physician regarding whether it is safe to crush the medication. If safe, medication should be crushed as fine as possible (into powder form) and dissolved into water before channeling medication through device. Channeling solid medication through the tubing that is not properly crushed can lead to blockage in the tubing. Never crush enteric coated medication or mix medication with formula. Flush device with water after channeling medication.

Flushing: Use room temperature water for tube flushing. The amount of water will depend on the patient's needs, clinical condition, and type of tube, but the average volume ranges from 10 to 50 ml for adults, and 3 to 10 ml for infants. Flush the feeding tube with water every 4-6 hours during continuous feeding, anytime the feeding is interrupted, before and after every intermittent feeding. Flush the feeding tube before and after medication channeling and between medications. Do not use excessive force to flush the tube. Excessive force can perforate the tube and can cause injury to the gastrointestinal tract.

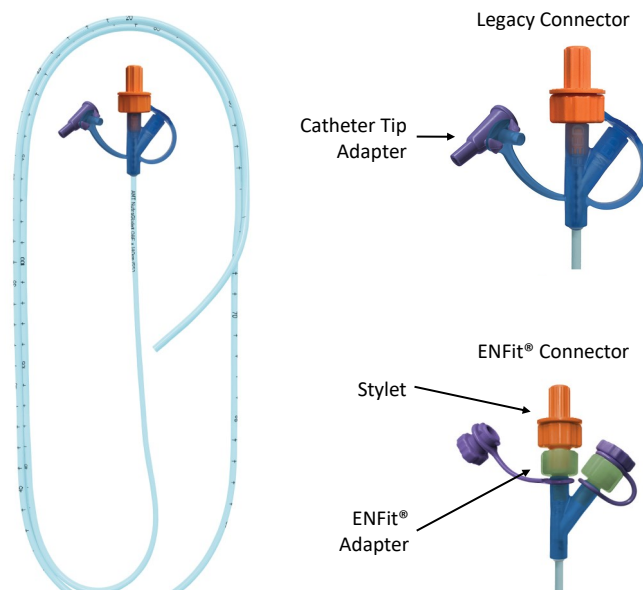
Figure 1: NutraGlide® Nasal Feeding Tube

How do you take care of the NutraGlide® after it has been placed?

Device Care: Check the device daily for any signs of damage or clogging. Clogging and/or reduced flow are indications of diminished performance. The feeding tube should be monitored, regularly assessed, and replaced when clinically indicated based on functionality and patient condition. The feeding tube should be monitored and regularly assessed for any resistance of flow and cleanliness of the port as well as a regular patient assessment for any nasopharynx trauma such as the occurrence of pain or the development of bleeding. If any of these symptoms are observed, contact your healthcare professional for advice.

Clogging: Connect a syringe filled with warm water to the Feed Port and gently push and pull the syringe plunger to free the clog. It may take several cycles of pushing / pulling the plunger to clear the clog. If clog cannot be removed, contact your healthcare professional, as the tube may need to be replaced. Do not use excessive force to flush the tube. Excessive force can perforate the tube and can cause injury to the gastrointestinal tract.

How long does the NutraGlide® last?

Nasal feeding tubes are meant to be periodically replaced for optimal performance, functionality, and cleanliness. The NutraGlide® has satisfied biocompatibility testing as a device for long-term use per ISO 10993-1. The feeding tube should be monitored, regularly assessed, and replaced when clinically indicated based on functionality and patient condition. The feeding tube should be monitored and regularly assessed for any resistance of flow and cleanliness of the port as well as a regular patient assessment for any nasopharynx trauma such as the occurrence of pain or the development of bleeding. Device performance and functionality can degrade over time depending on many factors, including: gastric pH, medications, trauma to the device, and overall tube care. Device should be replaced if signs of failure are noted.

What are the possible side effects of having the NutraGlide®?

Potential complications when using the NutraGlide® include but are not limited to: Patient discomfort • Nausea • Vomiting • Diarrhea • Pneumothorax • Gastrointestinal bleeding or ulceration • Gastrointestinal or esophageal perforation • Aspiration and/or aspiration pneumonia • Airway Obstruction • Tissue irritation or necrosis • Contamination

Please consult your healthcare professional if you experience any of the following: Nasopharynx trauma such as the occurrence of pain or the development of bleeding • Foul smell coming from the device.

There is a risk that the tube could become clogged or have reduced flow. The device may become discolored over days to months of use.

Is the NutraGlide® MRI Compatible?

The NutraGlide® is MRI Safe once stylet (if used) is removed. A patient with an indwelling device can be scanned safely.

How do you report adverse events?

- For Australian Users: Notice that any serious incident that occurs in relation to the device should be reported to Applied Medical Technology, Inc. and The Therapeutic Goods Administration (TGA) at <http://www.tga.gov.au>.
- Please contact AMT, our Authorized Representative (EC Rep), and/or the competent authority of the member state in which you are established if a serious incident has occurred in relation to the device.

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