
AMT Balloon Measuring Device

CE
2862

- ES** Dispositivo de medición con balón AMT.
DE AMT Ballonmessgerät.
IT Dispositivo di misurazione a palloncino AMT.
FR Dispositif de mesure à ballonnet AMT.
NO AMT Balloon Measuring Device.
SV AMT Balloon Measuring Device.
PT-BR AMT Balloon Measuring Device.
NL AMT-ballonmeetapparaat.
DA AMT-enhed til måling af ballon.
FI AMT Ballonki-mittauslaite.
AR AMT جهاز قياس بالون.
BG Балонно устройство за измерване AMT.
CS Balónkový zdravotnický prostředek AMT.
ET AMT balloonia mõõtmisseade.
EL Συσκευή Μέτρησης Μπαλονιού AMT.
HU AMT-ballonos mérőeszköz.
JA AMT バルーン測定デバイス.
LV AMT balona mērīšanas ierīce.
LT AMT balioninė matavimo priemonė.
PL Urządzenie pomiarowe AMT Balloon Measuring Device.
PT Aparelho de Medição com Balão AMT.
RO Dispozitiv de măsurare cu balon AMT.
SK Balónový zdravotnícky prostriedok AMT.
SL Pripomoček za merjenje z balonom AMT.
TR AMT Balonlu Ölçüm Cihazı.

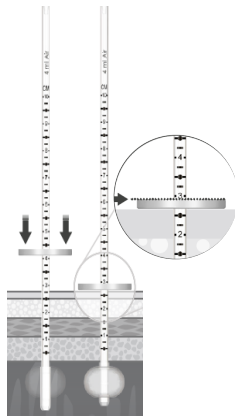STERILE EO

R_x ONLY

MD

CAUTION: Federal (USA) law restricts this device to sale, distribution, and use by or on the order of a physician.

INTENDED USE:

The AMT Balloon Measuring Device is intended for general measurement during a variety of medical procedures. It is also compatible with a guidewire. The AMT Balloon Measuring Device is intended to be used for patients of all ages (infant to elderly).

INDICATIONS FOR USE:

The AMT Balloon Measuring Device is a sterile, single use, manual measurement device intended to be used in various general procedures. Minimum French Size is 10FR.

CONTRAINDICATIONS FOR USE:

At this time there are no contraindications associated with the use of the AMT Balloon Measuring Device.

COMPLICATIONS:

At this time there are no complications associated with the use of the AMT Balloon Measuring Device.

NOTE: Please contact AMT, our Authorized Representative (EC Rep), and/or the competent authority of the member state in which you are established if a serious incident has occurred in relation to the device.

CLINICAL BENEFITS:

Clinical Benefits to be expected when using the AMT Balloon Measuring Device include but are not limited to:

- Device allows for precise anatomical measurements for choosing proper procedures, instruments and devices
- Aids in proper placement of guidewire compatible devices
- Reduces potential formation of granulation tissue and other skin issues resulting from improperly sized devices

PERFORMANCE CHARACTERISTICS:

Performance Characteristics of the AMT Balloon Measuring Device include but are not limited to:

- Provides a means of accurately measuring length or depth
- Helps to ensure proper fit, comfort, and safety of devices placed after measurement
- Guidewire Compatible

WARNING: THIS DEVICE IS INTENDED FOR SINGLE USE. DO NOT REUSE, RE-STERILIZE OR REPROCESS THIS MEDICAL DEVICE. DOING SO MAY COMPROMISE BIOCOMPATIBILITY CHARACTERISTICS, DEVICE PERFORMANCE AND/OR MATERIAL INTEGRITY; ANY OF WHICH MAY RESULT IN POTENTIAL PATIENT INJURY, ILLNESS AND/OR DEATH.

WARNING: THIS DEVICE IS TO BE USED BY TRAINED HEALTHCARE PROFESSIONALS AND SHOULD NOT BE USED CLINICALLY WITHOUT A FULL UNDERSTANDING OF THE DIRECTIONS FOR USE.

CAUTION: Please inspect all contents of the kit for damage. If the package is damaged or sterile barrier is breached, do not use the product. If any kit component is missing or damaged, discard and obtain another device.

INSTRUCTIONS:

1. Open the AMT Balloon Measuring Device package and inspect the parts. Verify balloon inflates prior to using. If any damage is noticed, discard and obtain another device for use.
2. Slide the measuring disc up the tube - away from the balloon. If the disc does not slide on the tube, discard and obtain another device for use.

NOTE: A guidewire can optionally be used to aid in guidance of the device. The AMT Balloon Measuring Device will slide over guidewires up to 0.038" in diameter.

NOTE: If the device is to be inserted into the body, lubricate the balloon tip using a water soluble lubricant, insert the tube, and inflate the balloon per step 3.


3. Insert a slip tip syringe containing 4ml AIR into the handle side orifice and inflate balloon. Remove the syringe.
4. Adjust the tube and disc so that the top of the balloon and bottom of the disc contact the surfaces between which measurement is desired.
5. Read the measurement value on the tube above the disc (side farther from balloon). The measurement accuracy of the device is +/- 0.2cm.
6. To deflate the balloon, place fingers as shown below. The thumb should be placed directly below inflation port, while index finger is placed directly above inflation port. Gently squeeze to remove air from balloon. A noticeable sound may be heard when air escapes.

NOTE: Squeezing too hard will block the valve port, preventing balloon deflation. Only a partial squeeze - enough to open the valve - is all that is needed.

NOTE: Typical deflation using a slip tipped syringe can also be used to deflate the balloon.

NOTE: The device can be disposed of by following local disposal guidelines, facility protocol or through conventional waste.

THANK YOU! Thank you for choosing AMT. For additional help and information regarding the use of our device, feel free to contact AMT with the contact information on the back of the instructions for use. We are happy to hear your thoughts and help with your concerns and questions.

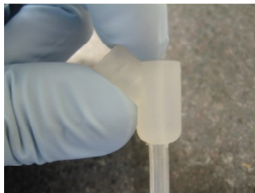Rx ONLY Prescription Only	STERILE EO Sterilized Using Ethylene Oxide
 Not made with natural rubber latex	 Not made with DEHP (Di(2-ethylhexyl) Phthalate)
 Do not resterilize	 Do not use if package is damaged and consult instructions for use
 Single Use Only	MD Medical Device

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© 2025 Applied Medical Technology, Inc.
Applied Medical Technology, Inc.
Made in the U.S.A.