AMT MiniSPC™ Drainage Set

CE
2862

- ES Juego de Drenaje MiniSPC™ AMT.
DE AMT MiniSPC™ Drainageset.
IT Kitt di Drenaggio AMT MiniSPC™.
FR Kit de Drainage MiniSPC™ AMT.
NO AMT MiniSPC™ Drainage Set.
SV AMT MiniSPC™ Drainage Set.
PT-BR Conjunto de Drenagem AMT MiniSPC™.
NL AMT MiniSPC™ Drainage Set.
DA AMT MiniSPC™-Drænagesæt.
FI AMT MiniSPC™ -Dreenisarja.
AR .AMT MiniSPC™ مجموعة الصرف
BG Дренажен Комплект AMT MiniSPC™.
CS Drenážní Sada AMT MiniSPC™.
ET AMT MiniSPC™ Drainage Set.
EL Σετ Αποστράγγισης AMT MiniSPC™.
HU AMT MiniSPC™ Drénkészlet.
JA AMT MiniSPC™ 排尿セット。
LV AMT MiniSPC™ Drenāžas Komplekts.
LT „AMT MiniSPC“™ Drenavimo Rinkinys.
PT Conjunto de Drenagem AMT MiniSPC™.
PL Zestaw do Drenażu AMT MiniSPC™.
RO Set de Drenaj AMT MiniSPC™.
SK Drenážna Súprava AMT MiniSPC™.
SL AMT MiniSPC™ Drenážni Komplet.
TR AMT MiniSPC™ Drenaj Seti.

MD

C4990-B 12/2025

STERILE EO

Rx Only

Made in the U.S.A.

FIG 1

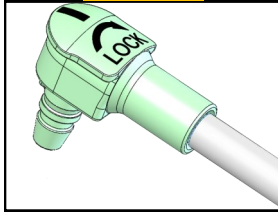
FIG 2 — MiniSPC™ Button


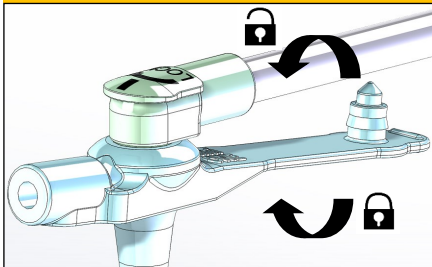
FIG 3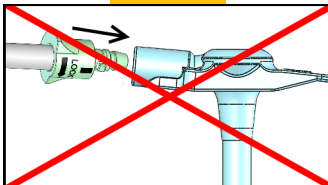

INDICATIONS, CONTRAINDICATIONS

INTENDED USE: The MiniSPC™ Button & Drainage Set provides a channel through which urinary bladder contents may be diverted directly from the bladder through a secured (initial placement) or formed (replacement) stoma. The device is intended to be used by clinicians and trained patients/caregivers.

INDICATIONS FOR USE: For patients 7 years of age and older with lower urinary tract dysfunctions such as neurogenic bladder dysfunction, congenital malformation or urethral obstruction, or who are severely disabled or requiring constant care. The MiniSPC™ Button & Drainage Set is indicated to be used for temporary suprapubic urinary diversion and drainage for less than or equal to four weeks. The MiniSPC™ Button is intended to be placed directly into the bladder through a secured (initial placement) or formed (replacement) stoma.

CONTRAINDICATIONS: At this time there are no contraindications for use of the AMT MiniSPC™ Drainage Set.

COMPLICATIONS: At this time there are no complications associated with use of the AMT MiniSPC™ Drainage Set.

NOTE: Please contact AMT, our Authorized Representative (EC Rep), and/or the competent authority of the member state in which you are established if a serious incident has occurred in relation to the device.

CLINICAL BENEFITS

Clinical benefits to be expected when using the Drainage Set include but are not limited to:

- Provides connection between urinary collection bags and suprapubic catheter device.
- Allows for channeling of fluids drained from the bladder.

PERFORMANCE CHARACTERISTICS

Performance characteristics of the Drainage Set include but are not limited to:

- Provides secure attachment to the suprapubic catheter device.
- Clamp provides additional leakage protection.

DRAINAGE SET INSTRUCTIONS

WARNING: THIS DEVICE HAS THE POTENTIAL TO MISCONNECT WITH SMALL BORE CONNECTORS OF OTHER HEALTHCARE APPLICATIONS. ONLY USE THIS DEVICE TO CONNECT TO COMPATIBLE DRAINAGE DEVICES.

WARNING: THE FUNNEL ADAPTER OF THE DRAINAGE SET HAS THE POTENTIAL TO MISCONNECT TO BREATHING SYSTEM, LIMB CUFF, AND NEURAXIAL CONNECTORS.

1. Prior to use, please inspect all contents of the kit for damage. If the package is damaged or sterile barrier is breached, do not use the product. Obtain another package.
2. The drainage set may be used for gravity drainage of the bladder.
3. Ensure the clamp is closed and attach the drainage set to the MiniSPC™ Button by lining up the dark line on the drainage set connector with the dark line of the interlock on the MiniSPC™ Button (Fig 1). Fully press drainage set connector to insert into the MiniSPC™ Button. Turn 3/4 to the right (clock-wise) to lock the drainage set into place (Fig 2).
4. Attach the funnel end of the drainage set to the urine collection device connector being used. Firmly insert the urine collection device connector into the funnel adapter while using a slight rotation to lock in place. Once connected, open the clamp to allow flow.

DRAINAGE SET INSTRUCTIONS

NOTE: For urine to flow away from the body, position the drainage adapter and collection bag such that it is always lower than the bladder. Improper elevation may result in unintended urine backflow.

WARNING: ONLY TIGHTEN BY HAND. NEVER USE EXCESSIVE FORCE OR A TOOL TO TIGHTEN THE CONNECTOR. IMPROPER USE CAN LEAD TO CRACKING, LEAKAGE, OR OTHER FAILURE.

WARNING: ENSURE DEVICE IS CONNECTED TO A DRAINAGE PORT ONLY AND NOT TO AN IV SET.

WARNING: IF THE DRAINAGE SET IS NOT PROPERLY PLACED AND LOCKED, LEAKAGE MAY OCCUR. WHEN TWISTING THE DRAINAGE SET CONNECTOR, HOLD THE MINISPC™ BUTTON IN PLACE TO AVOID ROTATION WHILE PLACING THE DRAINAGE SET. NEVER INSERT DRAINAGE SET CONNECTOR INTO THE BALLOON INFLATION PORT (FIG 3). THIS MAY CAUSE THE INTERNAL BALLOON TO DEFLATE OR LEAD TO BALLOON FAILURE IF IMPROPER CONTENTS ARE INSERTED INTO THE BALLOON.

5. When drainage is complete, remove the drainage set. Close the clamp to prevent leakage while removing the drainage set. Remove the drainage set by holding the MiniSPC™ Button in place, grasp the drainage connector, twist 3/4 turn to the left (counter clock-wise) to unlock (Fig 2), and gently remove the drainage set from the MiniSPC™ Button.
6. Flush the device in accordance with frequency and protocol established by your physician. A Luer slip syringe may be inserted directly into the drainage port of the MiniSPC™ Button for flushing. When flushing is complete, snap the MiniSPC™ Button plug in place to keep the lumen clean.
7. Store drainage set in a clean, dry place. A previously used drainage set may be reused for drainage only. It must not be used for flushing.

WARNING: IN CASE OF FEVER, ABDOMINAL DISTENTION, INFECTION, BLOCKAGE, TISSUE NECROSIS, SITE REDNESS, OR PURULENCE, PATIENTS SHOULD SEE THEIR PHYSICIAN IMMEDIATELY.

WARNING: DO NOT PLACE ANY FOREIGN OBJECTS INTO THE DRAINAGE PORT.

WARNING: DO NOT USE A PREVIOUSLY USED DRAINAGE SET FOR FLUSHING.

DEVICE LONGEVITY

Drainage sets are meant to be periodically replaced for optimal performance and cleanliness. AMT recommends that the drainage set be changed at least weekly or as often as indicated by your healthcare professional. Device performance and functionality can degrade over time depending on usage and environmental conditions. Some factors that can lead to reduced longevity include: frequency of use, trauma to the device, and overall drainage set care.

Device should be replaced if leakage, cracks, tears, excessive residue build-up, mold, fungus, or other signs of failure are noted. The device can be disposed of by following local disposal guidelines or through conventional waste. Some usage types can wear down device components quicker than others. If tubing is becoming hardened, replacement is recommended to avoid failure.

THANK YOU!

Thank you for choosing AMT. For additional help and information regarding the use of our device, feel free to contact AMT with the contact information on the back of the instructions for use. We are happy to hear your thoughts and help with your concerns and questions.

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© 2025 Applied Medical Technology, Inc.
**Applied Medical Technology, Inc.**